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Login Page

Please complete the form below in order to log in to our system. After that, you will be able to generate a PDF file from the data created by your Lady-Comp device.

Please register with your mail and a password. We will send a confirmation code to your email to verify the address.

E-mail

Password

Password

Other Pages

If you want to login onto your account, click here: [Login](#)

You don't have an account yet? Click here: [Register](#)

In case you lost your password, you can get a new one via email: [Lost password](#)

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Data Analysis

Login Page

Please complete the form below in order to log in to our system. After that, you will be able to generate a PDF file from the data created by your Lady-Comp device.

Please enter the activation code which has been sent to your e-mail.:

Activation code

Activate account

Information

Your account has been registered. Please check your mails in order to activate your account. In case you don't receive our email, please verify your spam folders.

Close

Other Pages

If you want to login onto your account, click here: [Login](#)

You don't have an account yet? Click here: [Register](#)

In case you lost your password, you can get a new one via email: [Lost password](#)

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Hello (il tuo indirizzo mail) Your account for Lady-Comp has been created. Please verify your account by copying the following code when you try to access it:

XXXX-XXXX

If You have problems, do not hesitate to contact our support at our website. Kind regards Your Lady-Comp team

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Edit your profile here.

Please complete your profile in case you have not done this yet. If you don't want to change your password, you can leave the two password fields empty.

You are able to edit your e-mail address. As this address is your login to your account, you have to verify your account after changing the address. In that case we would send you an e-mail.

Please click into the boxes in order to change their data.

E-mail	<input type="text" value="james.demo@ladycomp.it"/>
First name	<input type="text" value="Demo"/>
Last name	<input type="text" value="User"/>
New password	<input type="password" value="*****"/>
New password	<input type="password" value="*****"/>

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Edit your profile here.

Please complete your profile in case you have not done this yet. If you don't want to change your password, you can leave the two password fields empty.

You are able to edit your e-mail address. As this address is your login to your account, you have to verify your account after changing the address. In that case we would send you an e-mail.

Please click into the boxes in order to change their data.

Information

Your profile was successfully updated. You may now generate PDF files from the data of your Lady-Comp device.

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Get the visualisation of your Lady-Comp data here.

As soon as you have uploaded a file, you can download the generated output as PDF.

No file uploaded yet.

Upload a newer file to your account.

First, you need to get your the Lady-Comp file (usually *.txt) to your computer by connecting your device to it. Then, upload the file to your account by using the following form.

Click on the "Browse" button in order to upload the file. If you like, you may add a short comment.

File	<input type="button" value="Choose File"/> no file selected
Comment	<input type="text"/>

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Information

Please wait until we have processed your data. This could take a few seconds. The output starts as soon as you click "continue".

Information

Your output file has been successfully generated. You may download the PDF file now by clicking the box above. Please note that we will not store this file permanently!

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